


INSTALLING A LEARNING MODULE IN MÜSE

Prior to installing a CAE Healthcare Learning Module, ensure the Learning Module files are on an external storage device or the Instructor Workstation's hard drive.

IMPORTANT: You must have established a connection with your simulator prior to installing any Learning Module.

Access Müse

1. Log into the Müse software using an account with the appropriate privileges.
The home page appears.
2. Click the **System Administration** button in the top, right-hand corner.
The System Administration screen appears.

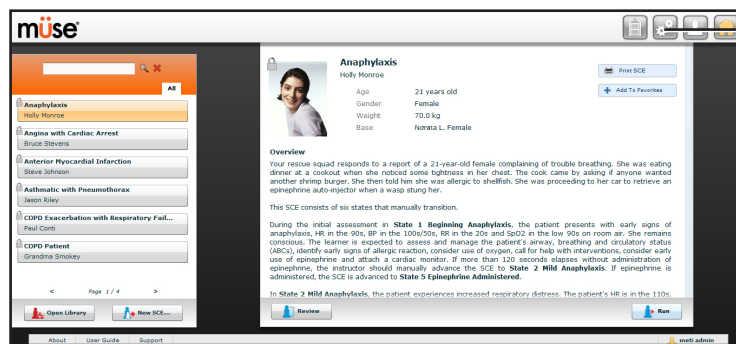

The **System Administration** button

The Home Page

3. From the System Administration screen, click **Content Management**.

The **Content Management** button



The Learning Module Panel

When the Content Management button is selected, the Learning Modules panel appears by default. If another panel has been selected, return to the Learning Modules panel by clicking the **Learning Modules** link.


From the Learning Modules panel, Learning Modules can be installed or deleted.



To Install a Learning Module in Müse

1. Click **Install Learning Module**.


The Select file to upload dialog box appears.

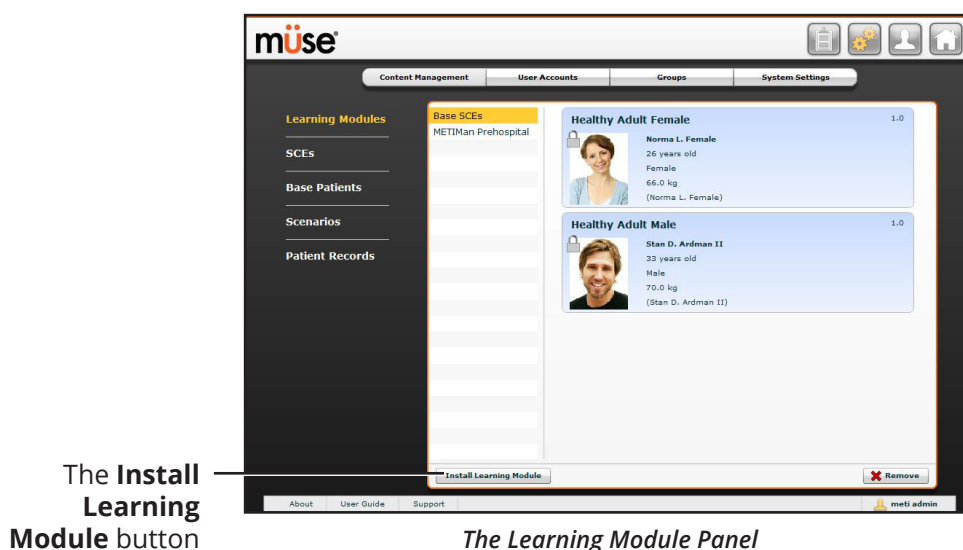

2. Locate the correct Learning Module file on the external storage device or the hard drive location where the Learning Module file is saved.

The file extension is **.mlm**.


3. Select the file and click **Select** or **Open**.

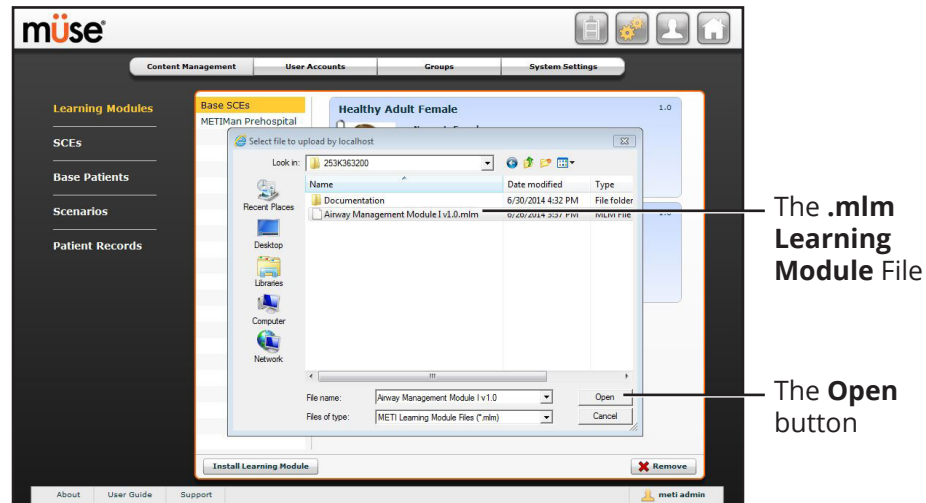

The Learning Module Panel

- Refresh the screen by clicking the **Home** button in the Müse software and then return to the Learning Modules panel.

The Learning Module appears on the Learning Modules panel and is available for use.

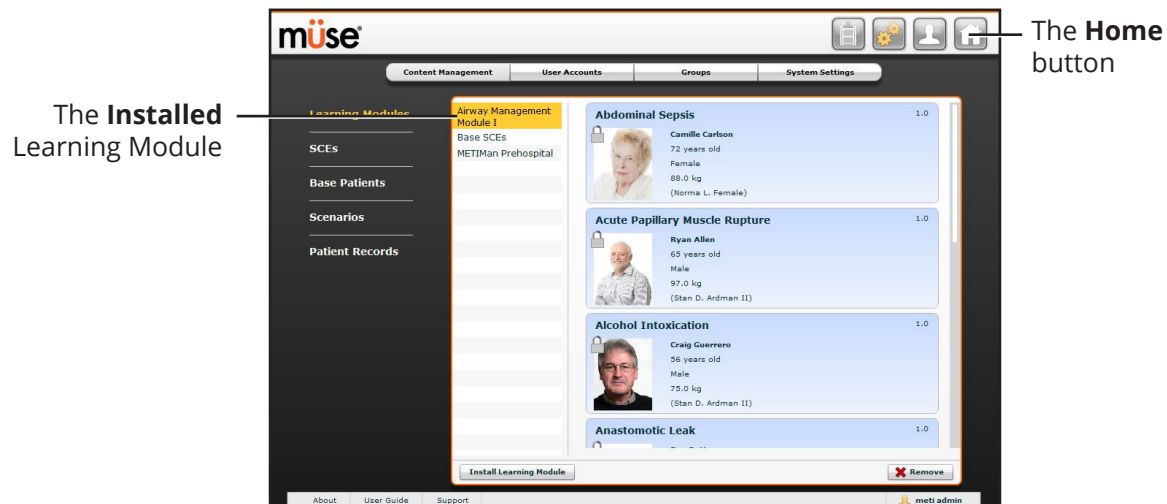


The Learning Module Panel

To Delete a Learning Module from Müse

1. Select a Learning Module from the Learning Module panel.

Learning
Module to
Remove

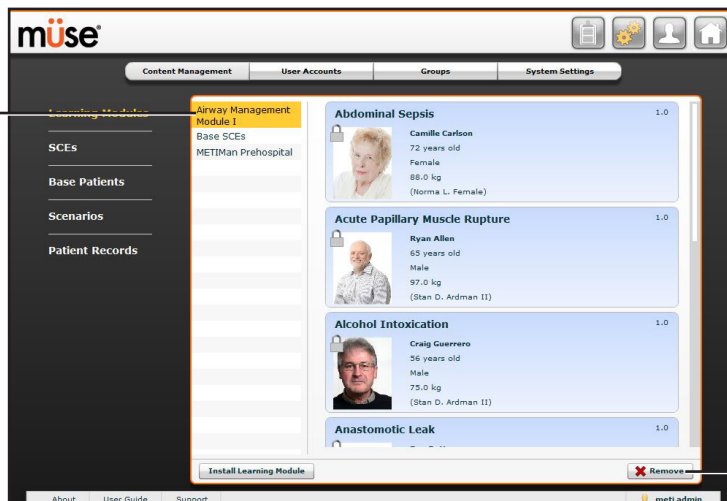

The **Remove**
button

The Learning Module Panel

2. Click the **Remove** button.

The Remove Learning Module warning appears.



The Learning Module Panel

3. Click **Delete**.

The Learning Module and all its SCes are deleted.

NOTE: Preconfigured Learning Modules and Base SCes cannot be deleted. If a user attempts to delete them, a failure message appears.